

SKINTASTIC AESTHETICS

SR-Wo8

Hydro Facial System Fourteen Handpieces & PDT Light Therapy

CPD ACCREDITED TRAINING INCLUDED

PROFESSIONAL COSMETIC FACIAL SYSTEM

Fourteen handpieces and a PDT light therapy lamp in one trolley.



SkinTastic
Aesthetic UK

Established Since 2011

Skintastic Aesthetic Supplies has been supplying aesthetic and laser systems, and delivering regulated training, from Southampton since 2011. We are an Ofqual regulated training academy, a supplier of professional treatment equipment, and a working clinic. Every machine we sell is supported by the people who use that technology every day.

Ofqual regulated academy

Delivering Focus Awards qualifications

CPD accredited training

Practical training included

Technical support

From Southampton

FCA authorised

For credit broking as a secondary activity

Full Member

British Medical Laser Association

 **This longevity reflects stability, governance and experience within the UK aesthetic sector.**



Fourteen Handpieces, One Platform

The SR-W08 brings hydro dermabrasion, radiofrequency, ultrasound and microcurrent, plasma, oxygen therapy and PDT light therapy together on one trolley. Manufacturer's model reference SR-W08, with PDT lamp model LB594B.

A therapist can build a complete facial from cleanse through to light therapy without the client leaving the couch.

1

Hydro Dermabrasion

8 tips, vacuum to 100 kPa

2

Radiofrequency

1 MHz, eye and face heads

3

Ultrasound, Microcurrent & Skin Scrubber

Three modalities in one step

4

Plasma Shower & Oxygen Therapy

Cleansing and clarifying

5

PDT Light Therapy Lamp

Model LB594B, 1,098 LEDs



Cleansing and Preparation

Four handpieces dedicated to preparing the skin — lifting debris, loosening congestion and creating the ideal base for the treatment steps that follow.

Hydro Dermabrasion Pen

Water and oxygen with warm bubbles, combined with adjustable vacuum, to cleanse the skin and lift debris from the pores. Eight tips supplied, four hard and four soft. Vacuum up to 100 kPa, warm water across ten levels.

Skin Scrubber

High frequency vibration used to lift dead skin cells and loosen congestion. Three modes, energy across ten levels.

Oxygen Bubble Pen

Produces fine bubbles from the D bottle for a gentle cleanse.

Plasma Shower

Ionises the air between the head and the skin, used as part of a cleansing and clarifying step. Two heads supplied.



FIRMING AND STIMULATING

Firming and Stimulating

Four handpieces focused on stimulating collagen, tightening the skin and improving tone across the face, jawline and eye area.

Bio Rollers

A pair of rollers delivering gentle electrical stimulation across the face and jawline. Two handles, energy across ten levels, three operating modes.

Bipolar Radiofrequency, Eye

A small radiofrequency head shaped for the delicate skin around the eye. 1 MHz, energy across ten levels.

Multi-Polar Radiofrequency, Face

A larger radiofrequency head for the cheeks, jaw and neck. 1 MHz, energy across ten levels.

Photon Bio Brush

Red light combined with microcurrent, worked over the face and neck. Both across ten levels.



Product Delivery and Finishing

Six handpieces for working products into the skin, delivering targeted serums, oxygen and thermal conditioning at the close of a treatment.

Ultrasound Handle

Ultrasonic vibration used to work products into the skin during a facial. Three modes, energy across ten levels.

Mesotherapy Gun

Combines radiofrequency, EMS and a micro-nanocrystal tip to help deliver serums into the upper layers of the skin. Five nanocrystal tips supplied.

Oxygen Pen

Delivers a fine spray of saline or purified water with oxygen.

Oxygen Mask

A dome mask that sits over the face to deliver a high concentration of oxygen, above 93% at the mask.

Hot and Cold Hammer

A metal head that heats or chills, used to warm the skin before treatment and to soothe it afterwards. Cooling minus 10 degrees C to plus 10 degrees C, heating 36 degrees C to 50 degrees C.



PDT Light Therapy Lamp

Model LB594B. A separate lamp with 1,098 LEDs on an adjustable arm, used at the end of a facial while the client rests.



650 nm Red

Skin rejuvenation and collagen support



535 nm Green

Pigmentation and skin tone



417 nm Blue

Acne and sebaceous regulation



Hybrid Mode

Combined wavelength output

 **Protective eyewear is supplied and must be worn by the client and the therapist throughout light therapy.**



Technical Specification

Specifications as declared by the manufacturer and correct at the time of publication.

Handpieces	14, including the PDT lamp
Hydro dermabrasion tips	8, four hard and four soft
Vacuum	Up to 100 kPa
Temperature range	Minus 10 °C to plus 50 °C
Radiofrequency	1 MHz
PDT lamp	Model LB594B, 1,098 LEDs
PDT light modes	650 nm red, 535 nm green, 417 nm blue, hybrid
Oxygen concentration	Above 93% at the mask
Power consumption	250 W
Packed dimensions	67 × 49 × 37 cm, plus the PDT lamp
Packed weight	39 kg, plus 5.5 kg for the PDT lamp



What's Included With Your SR-Wo8

Everything you need to start treating from day one — equipment, training and ongoing support.

1

The Equipment

Main machine, hydro dermabrasion pen with 8 tips, 2 bio rollers, skin scrubber, ultrasound handle, bipolar RF handle for the eye area, multi-polar RF handle for the face, hot and cold hammer, photon bio brush, mesotherapy gun with 5 nanocrystal tips, PDT therapy lamp, oxygen bubble pen, oxygen pen, plasma shower with 2 plasma heads, power cord and user manual.

2

The Training

CPD accredited training is included in the price and can be delivered online or in person at our Southampton academy. Training covers set-up, the treatment protocols for each handpiece, client consultation, contraindications and aftercare.

3

The Support

Telephone and email support throughout. Replacement consumables such as tips, bottles and solution can be ordered directly from us.

4

Warranty

One year on the main unit. Consumable parts and accessories are excluded.

5

Your Logo, At No Extra Cost

Your own logo can be designed on the screen and shell of the machine at no additional charge — a professional, branded finish from day one.



Investment

1

With Training — £2,999 excluding VAT

Machine plus CPD accredited training included in the price. Training can be delivered online or in person at our Southampton academy.

2

Machine Only — £2,699.10 excluding VAT

A 10% discount for qualified therapists who do not need the training.

Ways to Pay

Card, Apple Pay or Google Pay at checkout, or spread the cost with Klarna or Clearpay. Business finance is also available.

- ☐ Skintastic Aesthetic Supplies Ltd is authorised and regulated by the Financial Conduct Authority for credit broking as a secondary activity. We are a credit broker, not a lender. Finance is subject to status, affordability checks and acceptance by the lender.



THE SKINTASTIC DIFFERENCE

Why Buy From Skintastic?

We are not a box shifter. We train on this equipment, we treat on this equipment, and we support it.

Fifteen years supplying professional aesthetic systems

Ofqual regulated academy, not a weekend certificate

CPD accredited training included in the price

Technical support from Southampton, not an overseas call centre

Consumables and spares available directly from us

Honest documentation about what a machine is, and what it isn't

 **Clinically Engineered. Commercially Powerful. Exclusively Skintastic.**



Important Information

Device Classification

The SR-W08 is supplied as a **cosmetic instrument for aesthetic facial treatments. It is not a medical device, it is not MHRA registered as one, and it should not be used for medical procedures or presented as medical treatment.**

Safe Use

Protective eyewear is supplied and must be worn by the client and the therapist during light therapy. Treatments must be carried out only by trained operators, following consultation and, where appropriate, a patch test. Follow the manufacturer's instructions for use for contraindications and operating limits.

Results

Results vary between individuals and no outcome is guaranteed.

Pricing

All prices exclude VAT and are correct at the time of publication.



Pricing and Purchase Routes

Two ways to buy the SR-W08, depending on whether you need the training. Both include the machine, engineer support and the one year warranty on the main unit.

ROUTE ONE · MOST POPULAR

Complete Package

£2,999

PLUS VAT

The machine with CPD accredited training from our Ofqual regulated academy, delivered online or in person to suit you, and engineer support from Southampton.

Training included

ROUTE TWO

Machine Only Purchase

£2,699.10

PLUS VAT

Save 10%

For therapists who already hold the relevant qualification and do not need the training package. The machine, engineer support and the warranty are all included.

Training not included

Warranty. **One year on the main unit.**

Please note. **The SR-W08 is a cosmetic device. It is not a medical device and it is not MHRA registered as one.**

Finance available, **subject to status.** Skintastic Aesthetic Supplies Ltd is authorised and regulated by the Financial Conduct Authority, FRN 964293, with secondary credit broking permission.

Talk to Us

Skintastic Aesthetic Supplies Ltd
Raising standards in advanced aesthetics since 2011.

Visit Us

26 The Avenue, Southampton, SO17 1XL

Telephone

02381 946394

Email

info@skintasticaesthetics.com

Website

www.skintasticaesthetics.com

Registered in England and Wales no. 11028743. VAT no. 368 8599 17. FCA reference 964293.

