

SKINTASTIC AESTHETICS

5-in-1


**Ultimate Body Contour System
Five Technologies. One Trolley.**

COSMETIC BODY CONTOURING SYSTEM

FOR PROFESSIONAL USE ONLY

Five contouring technologies. One trolley. One operator.

SkinTastic
Aesthetic UK

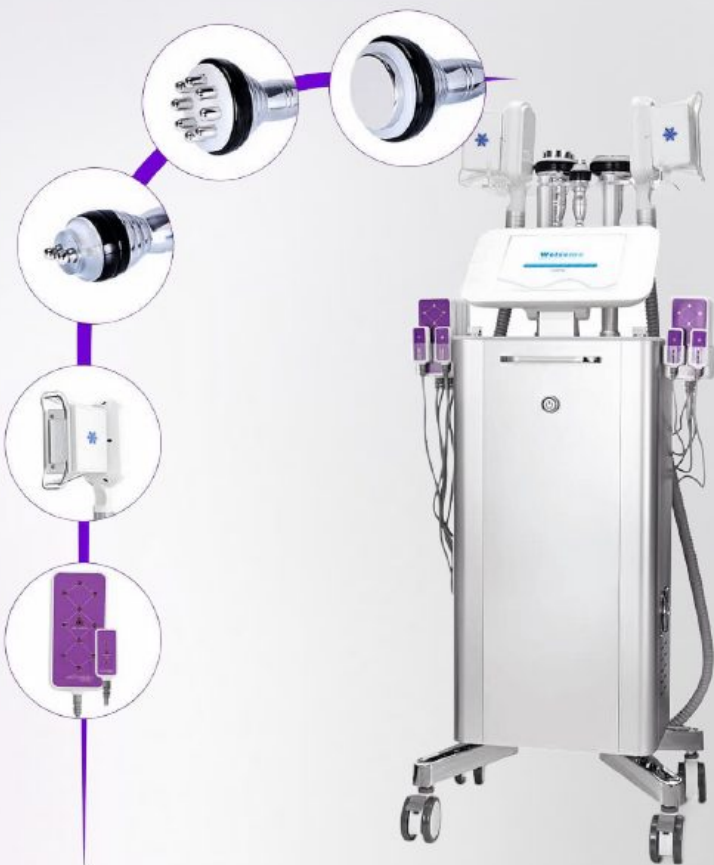

Established Since 2011

Skintastic Aesthetic Supplies has been supplying aesthetic and laser systems, and delivering regulated training, from Southampton since 2011. With over a decade of operational experience, the company is recognised for:

Ofqual regulated academy

CPD accredited training

UK Responsible Person (MHRA)

FCA authorised finance introductions

Full BMLA membership

- ☐ **Every machine we sell is supported by the people who use that technology every day, from a working clinic and training academy in Southampton.**



Five Technologies, One Platform

The **Ultimate 5-in-1 Body Contour System** consolidates five separate treatment systems into a single trolley. Manufacturer's model reference MS-7005F. Each system works independently or in sequence across one appointment, so a single operator can deliver a complete body contouring treatment without moving the client between machines.


Cryolipolysis Vacuum

40 kHz Ultrasonic Cavitation

Six-Polar Body Radiofrequency

Facial & Eye Radiofrequency

Low-Level Laser Pads

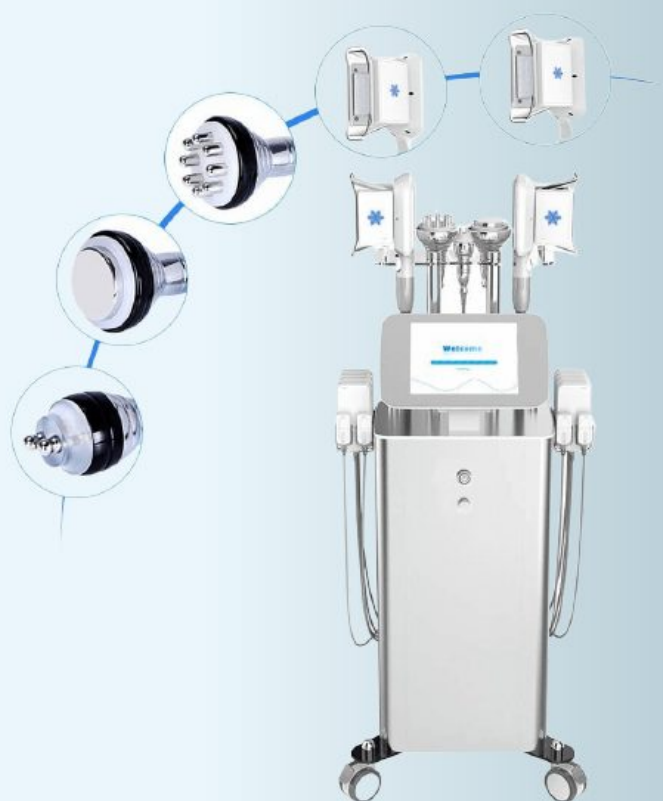

- ❏ **One machine, five treatment systems. Designed to consolidate infrastructure while maximising treatment capability and revenue potential from a single appointment.**



TREATMENTS AVAILABLE WITH THE ULTIMATE 5-IN-1

A complete body and facial contouring menu from one platform.

Abdomen contouring

**Waist and flank
shaping**

Upper arm treatment

**Thigh and buttock
contouring**

Back treatment

**Facial contouring and
jawline definition**

**Periorbital treatment
around the eye area**

**Cavitation followed by
radiofrequency
combination protocols**

**Cryolipolysis with
laser pad combination
protocols**

**Full-session body
sculpting sequences**

 Expand your treatment menu without investing in multiple standalone systems.



Cryolipolysis Vacuum

Controlled cooling with vacuum draw, applied to a targeted area. The applicator is designed to draw and hold the treatment area throughout the session.

Cooling Output

+5 °C to -5 °C

Vacuum

0-5 kPa

Power

Up to 200 W

Cooling Medium

Pure water

☐ An antifreeze membrane must always be applied before treatment. Never omit this step.



40 kHz Ultrasonic Cavitation

Ultrasonic handpiece for larger body areas. Suited to the abdomen, flanks, thighs and upper arms.

Frequency

40 kHz

Power

Up to 60 W

Tip Diameter

60 mm

- ☐ **Not to be used over the head, chest, breast, heart or back. Operator must follow the contraindication guidance in the instructions for use.**



Six-Polar Body Radiofrequency

Six-polar handpiece for skin tightening across the body. Commonly worked immediately after cavitation within the same session, as part of a combined contouring and tightening protocol.

Frequency

3 MHz

Power

Up to 30 W

Electrode Configuration

Six-polar

- ❏ **Pairing cavitation with radiofrequency in the same session is one of the most productive combination protocols available on this platform.**



Facial & Eye Radiofrequency

A smaller handpiece extends the machine beyond body work. Sized for facial contouring and the periorbital area, this is what turns a body machine into a full-day treatment room asset, letting you offer facial appointments between body sessions.

Frequency

3 MHz

Power

Up to 30 W

Application

Facial contouring and the periorbital area

 **Keep all handpieces away from the eyes and the thyroid. Always start at the lowest energy setting and increase gradually.**



Low-Level Laser Pads

Twelve pads, strapped in place, leaving the operator free while the client is treated. Positioned across the abdomen, flanks, arms, thighs and back according to the treatment plan.

Pad Configuration

- **8 large pads**, 10 diodes each
- **4 small pads**, 2 diodes each
- Total: 12 pads supplied

Technical Parameters

- Wavelength 635–650 nm
- 5 mW per diode

📄 The operator is free throughout the laser pad phase. Position the pads and use the time productively while the client is treated.



Client Selection

Professional practice starts with knowing who not to treat. The following clients must not be treated with any function of this system.

**Pregnancy,
breastfeeding or
menstruation**

**Heart disease,
pacemaker, or
implanted metal in the
treatment area**

**Hypertension, diabetes
or thyroid disease**

**Malignancy,
haemophilia or any
bleeding disorder**

Epilepsy

**Broken skin, wounds,
skin disease or
infectious disease**

**Recent surgery,
implants, fillers or
wrinkle-relaxing
injections in the area**

**Known metal allergy
or highly sensitive skin**



Operating Safely

Taken from the manufacturer's instructions for use. This is a starting point, not a substitute for your own consultation and record keeping.

Never use 40 kHz cavitation over the head, chest, breast, heart or back



Keep all handpieces away from the eyes and the thyroid

Always apply an antifreeze membrane before the cryolipolysis applicator

Start at the lowest energy setting and increase gradually

Remove all metal jewellery before treatment

Maintain full skin contact to avoid uneven heating

Technical Specification

Complete specifications across all five platform technologies.

Cryolipolysis Vacuum

Output temperature	+5 °C to −5 °C
Vacuum	0–5 kPa
Power	Up to 200 W
Cooling medium	Pure water

40 kHz Ultrasonic Cavitation

Frequency	40 kHz
Power	Up to 60 W
Tip diameter	60 mm

Six-Polar Body Radiofrequency

Frequency	3 MHz
Power	Up to 30 W
Electrode configuration	Six-polar

Facial & Eye Radiofrequency

Frequency	3 MHz
Power	Up to 30 W

Low-Level Laser Pads

Wavelength	635–650 nm
Output per diode	5 mW
Large pads	8 pads, 10 diodes each
Small pads	4 pads, 2 diodes each

System

Rated input voltage	AC 220–240 V, 50 Hz
Rated input power	350 VA
Mobility	Floor-standing trolley on castors
Manufacturer model reference	MS-7005F

Specifications as declared by the manufacturer and correct at the time of publication.

What's Included With Your Ultimate 5-in-1

Your system is supplied with everything you need to start treating, supported by people who use this equipment every day.

1

The Equipment

- Main unit on castors
- Cryolipolysis vacuum handle
- 40 kHz cavitation handle
- Six-polar body RF head
- Eight large laser pads, four small laser pads
- Holders, straps and power cord

2

The Training

- CPD accredited training from our Ofqual regulated academy
- Consultation and client selection
- Contraindications
- Treatment protocols
- Aftercare guidance

3

The Support

- Ongoing technical support from Southampton
- Guidance on marketing your new treatments
- Warranty and aftercare terms confirmed in writing before you commit



CUSTOM BRANDING

Your Logo on the Machine

Your own logo can be applied to the machine shell and screen at manufacture.

Photographs Better

A machine carrying your clinic's branding photographs better and reinforces your brand in every social post.

Looks Considered

Branded equipment looks considered rather than generic in the treatment room, and signals professionalism to every client.

Made to Order

Please allow additional production time for branded units. Discussed and confirmed before your order is placed.



Investment & Finance Options

Complete five-technology body contouring system. Free UK delivery on orders over £99.

£2,999 + VAT

Finance available. Indicative payments from around £108 per month over a three year lease for an established business, subject to status.



1

Equipment Price

£2,999 excluding VAT. Free UK delivery on orders over £99.

2

Finance Available

Arranged through Kennet Equipment Leasing Limited. Figures are indicative only and are not a quotation. Rates depend on your trading history and the lender who approves your application, and are confirmed in writing before you commit. A documentation fee applies.

Warranty and aftercare terms are confirmed in writing before you commit. Finance rates and indicative figures are confirmed in writing before you commit.

Why Buy From Skintastic?

We are not a box shifter. We train on this equipment, we treat on this equipment, and we support it.

Fifteen Years Supplying Professional
Aesthetic Systems

Ofqual Regulated Academy, Not a
Weekend Certificate

CPD Accredited Practical Training

UK Responsible Person (MHRA)

Technical Support from Southampton,
Not an Overseas Call Centre

Full BMLA Membership

FCA Authorised Finance Introductions

Honest Documentation About What a
Machine Is, and What It Isn't

Finance Arranged With No Commission
Taken by Skintastic

 **Five contouring technologies. One trolley. One operator. Backed by the people who use that technology every day.**



IMPORTANT INFORMATION

Important Information

Device Classification

The Ultimate 5-in-1 Body Contour System is a **cosmetic body contouring system for professional use. It is not a medical device and carries no medical device registration.**

Results

Results vary between individuals and no outcome is guaranteed. Treatments must be carried out only by trained operators, following consultation and, where appropriate, a patch test.

Pricing

All prices exclude VAT and are correct at the time of publication.



Pricing and Purchase Routes

The Ultimate 5-in-1 Body Contour System, supplied complete with CPD accredited training from our Ofqual regulated academy and technical support from Southampton.

COMPLETE SYSTEM · ONE PRICE

Ultimate 5-in-1 Body Contour System

£2,999

PLUS VAT

Five body contouring technologies on one trolley:
cryolipolysis vacuum, 40 kHz ultrasonic cavitation, 3 MHz six
polar body radiofrequency, facial and eye radiofrequency,
and 635 to 650 nm low level laser pads.

Supplied complete, with nothing further to buy to start treating

CPD accredited training, included

Training from our Ofqual regulated academy, online or in person, so the people using the system understand the technology, the contraindications and the treatment protocols before they treat a client.

Technical support, included

Support from Southampton rather than from an overseas manufacturer, with consumables and replacement parts supplied directly by us.

Please note. **This is a cosmetic device. It is not a medical device and it is not MHRA registered as one.**

Contraindications **and the full specification are set out earlier in this brochure. Please read them before you order.**

Finance available, **subject to status. Skintastic Aesthetic Supplies Ltd is authorised and regulated by the Financial Conduct Authority, FRN 964293, with secondary credit broking permission.**

Skintastic Aesthetic Supplies Ltd

Five contouring technologies. One trolley. One operator.

Skintastic Aesthetic Supplies Ltd, raising standards in advanced aesthetics since 2011.

Visit Us

26 The Avenue, Southampton, SO17 1XL

Telephone

02381 946394

Email

info@skintasticaesthetics.com

Website

www.skintasticaesthetics.com

Skintastic Aesthetic Supplies Ltd is registered in England and Wales, company number 11028743, registered office 26 The Avenue, Southampton SO17 1XL. VAT registration number 368 8599 17. Authorised and regulated by the Financial Conduct Authority, firm reference number 964293.

We act as a credit broker, not a lender. Finance is subject to status and individual lender approval.

